

PROETCH™



Etching Gel – Phosphoric acid 37%

ProEtch contains phosphoric acid for etching tooth and dentin when preparing surfaces for the application of composites.

Ingredients

Phosphoric acid	37%
Water and Filler	63%

Features

- High quality, non-drying, medium viscosity gel
- Stays put, does not run.
- Large, comfortable to use syringe
- Tip can be angled to facilitate easy and accurate placement of the gel even in distal & lingual composite preparations.
- Washes off quickly and easily
- Has excellent water solubility
- Visible color
- Disposable tips save time and are more sanitary

Recommended Procedure

ProEtch can be applied precisely in controlled amounts to the tooth enamel and dentin area to be etched.

The effective recommended time is:

Enamel – 30 to 60 seconds

Dentin – As indicated by the manufacturer of the bonding agent.

The Gel Etch should be rinsed off with copious amounts of water for at least 20 seconds.

Drying

Enamel – Dry adequately until the surface has a frosty white appearance.

Dentin – As indicated in the instruction for use of the bonding agent.

**The disposable tip should be removed and discarded after use!
Recap syringe after each use to prevent drying of the etch gel.**

Caution: For Dental Use Only

ProEtch™ contains phosphoric acid.

Instructions for Use should be carefully read prior to using ProEtch™.

In case of allergic reactions immediately stop the application and advise the patient to consult a physician.

If accidental contact with eyes occurs, flush immediately with a large amount of water.

Storage

Store ProEtch™ in a cool place at 24°C (75°F)

Shelf life indicated on product label.

Do not use after expiry date.

Warranty

Silmet Ltd. will replace product that is proven to be defective.

Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.



Silmet Ltd.

12 Hassadna st. Or-Yehuda 6022011, ISRAEL

Tel: 972-3-7353000 Fax: 972-3-5331581

info@silmet.co.il, www.silmetdental.com



Diadent

2 bis chemin du Loup, 93290 Tremblay en-France

Tel: 33-1-48012710 Fax: 33-1-42465230

68PE.Rev2 05/2015

SILMET